

WBC DIFF POCT

August Checks



Health Centre:

✉ wbcdiff@flinders.edu.au

☎ 08 8201 7555

Completed by:

Date:

Stock Levels

WBC DIFF microcuvettes

Microcuvette expiry date(s)
YYYY MM DD on box

DIFF SAFE dispensers

Cleaner Plus sticks

Order Placed

NT Health – TE POC order form/TE PPN

ACCHSs – Radiometer Pacific

- WBC DIFF Microcuvettes
- DIFF SAFE dispensers
- Cleaner Plus sticks
- Quality control solutions
(if applicable)

Device Care & Cleaning

- Cuvette holder - week 1
- Cuvette holder - week 2
- Cuvette holder - week 3
- Cuvette holder - week 4/5
- Optical parts
- Submit error code report

Quality Tests

Date bottle opened:

10-week expiry date:

- Quality control level 1: HD05221
- Quality control level 2: HD05222
- Quality control level 3: HD05223

Result Transmission

- Turn on your WBC DIFF device & plug it into an active internet port for result transfer

n/a if your device is always connected to the internet

Operator Training

Ask new staff to complete & submit an Operator training application form.