


HemoCue WBC DIFF Client Testing



1 Prepare device

Pull cuvette moving arm so cuvette holder is visible.


Ensure power adapter, keyboard and printer are plugged in.

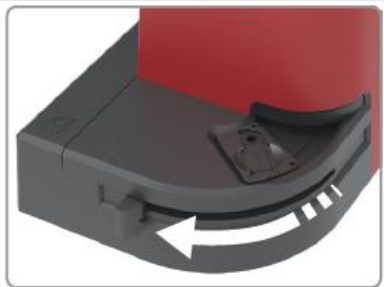

2 Turn device on

Press & hold the left button until the display is activated.

A window will show software version ~15 seconds.

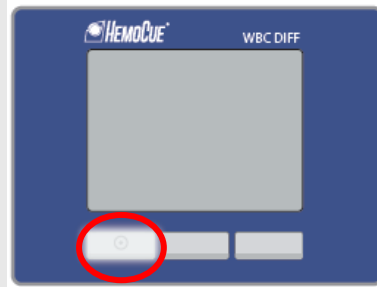

3 Printer ready

Press M to start printer.

Printer is ready when green light is steady.



4 Select 'Patient Test'

Press the middle button below the cuvette symbol to select *Patient Test*.




5 Enter ID Info

Enter required data:
1. Operator ID
2. Patient ID
3. Site ID

Confirm if correct. ✓

Insert cuvette window will be displayed.

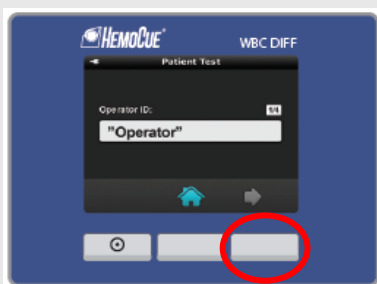

6 Mix blood

Gently **invert** blood collection tube* **20 times** immediately before dispensing.

*Must be EDTA tube



7 Dispense blood

Insert DIFF SAFE dispenser into tube.

Invert, gently push down (~2-3 secs) to dispense a drop of blood onto inside of microcuvette wrapper.



8 Fill cuvette

Hold microcuvette on a 45° angle. Place tip into blood drop & **fill in one continuous process.**

DO NOT double dip or skim blood.



9 Wipe cuvette

Wipe excess blood from each side & bottom of microcuvette using lint-free tissue.

Avoid touching open (curved) edge.



10 Load cuvette

Place filled microcuvette into cuvette holder.


Gently push cuvette moving arm shut.



11 Record results

Results will be ready in **<5 mins.**
Press right button to accept results. ✓

Record data in patient file.

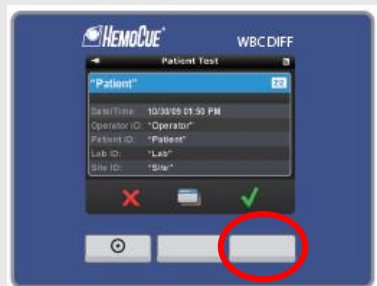

12 Dispose cuvette

Pull cuvette moving arm open. Remove microcuvette & dispose according to local guidelines for contaminated sharps.

