



# HemoCue WBC DIFF Review Results

1




Turn device ON & press right button below *Menu* icon.

2




Press right button to accept *Menu*.

3

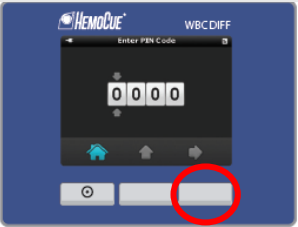


Enter PIN code 0000.  
Press right button to confirm.

4




Press middle button to enter *Review/Delete*.

5



Press middle button to enter *Review*.


6



The latest results are displayed.  
Use left & right buttons to move between past results.


WBC	10 <sup>9</sup> /L	DIFF	10 <sup>9</sup> /L	%
		NEU	6.5	33
		LYM	4.4	27
		MON	3.3	20
		EOB	2.2	13
		PLA	1.1	7

7



Press middle button to move between results & data window.  
Press left button to return to main menu.

8



While it is possible to view past results, it is not possible to print them.  
The device stores 600 results.